Instructions for Use

BISCO, INC.
1100 W. Irving Park Road
Schauamburg, IL 60193
U.S.A.
1-800-247-3688
1-847-534-5000
www.bisco.com

BISCO France
208, allée de la Couduette
13680 Lançon de Provence
France
33-4 90 42 92 92

IN-149R16
Rev.5/12

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed Dentist.
The CHOICE 2 kits containing etchant wBAC are not CE marked.

en CHOICE™2
GENERAL INFORMATION
CHOICE 2 LIC is a new, light-cured, high compression strength, light-cure luting cement that is exclusively formulated for cementing porcelain, lithium disilicate and zirconia veneers. CHOICE 2 is available in clear and white shades. CHOICE 2 white shade TRI-VENEER is available against an extra charge.

INDICATIONS FOR USE
Cementation of porcelain and zirconia veneers.
Retention on keratinized dentition.
Intended user: Licensed dentist.

Contraindications:
• Not for use with ALLOY UNIVERSAL™ or patients who have a history of severe allergic reactions to methacrylate resins.

Advantages:
• ALLOY UNIVERSAL, ALLOY BOND™ and „BIS-BLAND™“ are highly translucent.
• Hybridization is a very powerful and aggressive feature. It not only seals the resin into the pores of the porcelain, but also penetrates the micropores of the porcelain itself. This results in a permanent, high-strength bond between the resin and porcelain.
• Choice 2 is a very powerful and aggressive feature. It not only seals the resin into the pores of the porcelain, but also penetrates the micropores of the porcelain itself. This results in a permanent, high-strength bond between the resin and porcelain.
• Choice 2 is a very powerful and aggressive feature. It not only seals the resin into the pores of the porcelain, but also penetrates the micropores of the porcelain itself. This results in a permanent, high-strength bond between the resin and porcelain.
• Choice 2 is a very powerful and aggressive feature. It not only seals the resin into the pores of the porcelain, but also penetrates the micropores of the porcelain itself. This results in a permanent, high-strength bond between the resin and porcelain.

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