SELECT HV® ETCH High Viscosity

35% Phosphoric Acid Etchant

SELECT HV® ETCH (35%)

SELECT HV® ETCH (35%) w/BAC

en SELECT HV® ETCH 35% Phosphoric Acid Etchant

GENERAL INFORMATION

BISCO's new SELECT HV ETCH allows for precise selective placement. This etchant provides flowability, yet is just viscous enough to permit migration. Selective etching provides a deep phosphoric acid etch on the enamel margins while identifying the benefits of etch adhesives, such as low postoperative sensitivity and moisture tolerance on dentin. The visually brilliant coloring of this new line of etchants also makes the phosphoric acid etch for everyday etching where deep color differentiation is important.

BISCO's SELECT HV etchant is a phosphoric acid (H₃PO₄) intended for use on enamel, dentin and as a cleaning agent for restorative substrates. **SELECT HV ETCH** will effectively remove the smear layer, etch and demineralize enamel and dentin, reducing the necessary microretentive surface for successful bonding. The etchant rinses away cleanly and quickly, leaving no residue. **SELECT HV ETCH** is also available with benzalkonium chloride (BAC), an antimicrobial agent that inhibits the development of residual bacterial inhibition zones of up to 7 mm, when challenged (in-vitro) with *Actinomyces viscosus* *. **NOTE:** The inclusion of benzalkonium chloride has not been shown to correlate with a reduction in secondary decay in patients. In-vivo clinical studies to evaluate the effect of benzalkonium chloride on oral bacteria or caries have not been performed.

Indications For Use:

- Etching dentin and enamel
- Selective enamel etching
- Cleaning agent on dental restorative materials

Intended user: licensed dentist.

Warnings:

- The phosphoric acid in the etchant is a severe eye and skin irritant. Injury may result if the etchant is allowed to remain on the skin or mucosa for extended periods of time. In case of contact with other tissues, rinse immediately with plenty of water for several minutes.
- If accidentally splashed into the eye, flush with copious amount of water and seek medical attention immediately.

Caution:

- Do not mix with any other dental products.
- Product may contain items that are designed for one time use. Dispose of used or contaminated tips. Do not clean, sterilize or reuse.
- Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If syringe becomes contaminated, discard. Do not clean or disinfect.

Precautions:

- Keep away from direct sunlight. Prolonged exposure to sunlight could cause a discoloration of the etchant; this will not compromise the efficacy of the product.
- Test the application of Etchant on a mixing pad or a glass slide, BEFORE use on patients so as to become familiar with the pressure required to dispense the Etchant from the tip. Never use injector before checking flow from tip. Never use force to protrude to start flow intrarily; this could result in a subsequent uncontrolled discharge of material. If resistance to flow is experienced, DO NOT proceed. Replace with new tip and verify flow before use on patient.
- Propry pastes containing oil or fluoride should not be used as these additives may interfere with etching.
- See individual component labels for specific expiration dates.
- Keep this etchant away from children.
- Safety data sheet available at www.bisco.com.
- Instructions for Use available upon request.
- Instructions for Use available at www.bisco.com.

INSTRUCTIONS FOR USE

- Etching dentin and enamel**
 - Remove white cap from etchant syringe.
 - Attach the light blue 25 gauge lip on the 1.2ml syringe, or the dark blue 22 gauge lip on the 5g syringe.
 - Verify flow of etch before intralungal application. Dispense a small portion of gel onto a pad or gauze to familiarize yourself with its viscosity.
 - If surface to be etched is rough, use a pumice-water slurry. Propxy pastes containing oil or fluoride should not be used as these additives may interfere with etching. Rinse well.
 - Isolate the preparation.
 - Place etchant directly onto the surfaces to be etched. Leave etchant in place for 15 seconds.
 - Thoroughly rinse with water for 5 seconds with suction. If selectively etching enamel, suction prior to rinsing to avoid etchant flowing on to dentin.
 - Immediately proceed with the bonding procedure.
 - Discard all single use components.
- Selective enamel etching**
 - Remove white cap from etchant syringe.
 - Attach the light blue 25 gauge lip on the 1.2ml syringe, or the dark blue 22 gauge lip on the 5g syringe.
 - Verify flow of etch before intralungal application. Dispense a small portion of gel onto a pad or gauze to familiarize yourself with its viscosity.
 - If surface to be etched is not freshly cut, clean with a pumice-water slurry. Prophy pastes containing oil or fluoride should not be used as these additives may interfere with etching. Rinse well.
 - Isolate the preparation.
 - Place etchant directly onto the surfaces to be etched. Leave etchant in place for 15 seconds.
 - Thoroughly rinse with water for 5 seconds with suction. If selectively etching enamel, suction prior to rinsing to avoid etchant flowing on to dentin.
 - Immediately proceed with the bonding procedure.
 - Discard all single use components.

- Cleaning agent on dental restorative materials**
 - SELECT HV ETCH** may be used on restorative substrates to effectively remove organic contaminants. Apply etchant to the surface for 30 seconds and rinse thoroughly with water. **NOTE:** Unprimed zirconia surfaces should NOT be cleaned with phosphoric acid. Use a restoration cleaner such as ZiClean to remove contaminants.

- HYGIENE:** Re-cap syringes with the laser lock cap. Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended.
- DISPOSAL:** Refer to community provisions relating to waste. In their absence, refer to national or regional provisions relating to waste.
- STORAGE:** Store at room temperature (20°C/68°F - 25°C/77°F). Store away from direct sunlight.

WARNING: BISCO, Inc. recognizes its responsibility to replace products if proven to be defective. BISCO, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

- SELECT HV and ZiClean are registered trademarks of BISCO, Inc.
- Dr. Daniel Chan, University of Texas Health Science Center at San Antonio Dental/School, Residual/Effect 1 and 2% Benzalkonium Chloride/Incorporated into an Etchant on the Susceptibility of Actinomyces viscosus T14V, 1993

NOTE: Report any serious incident to BISCO and the competent authority of the state in which the user and/or patient is established.

TO REORDER: X-80611N – 125 light blue Disposable Tips (25 Gauge) X-80669N – 50 Dark Blue Disposable Tips (22 Gauge)

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- SELECT HV and ZiClean sind eingetragte Marken von BISCO, Inc.
- Dr. Daniel Chan, University of Texas Health Science Center at San Antonio Dental/School, Residual/Effect 1 and 2% Benzalkonium Chloride/Incorporated into an Etchant on the Susceptibility of Actinomyces viscosus T14V, 1993

NOTICE: Report any serious incident to BISCO and the competent authority of the state in which the user and/or patient is established.

TO REORDER: X-80611N – 125 light blue Disposable Tips (25 Gauge) X-80669N – 50 Dark Blue Disposable Tips (22 Gauge)

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