
# BISCO

## Adhesive Cmentation System For LITHUM Disilicate Restorations

## Instructions for Use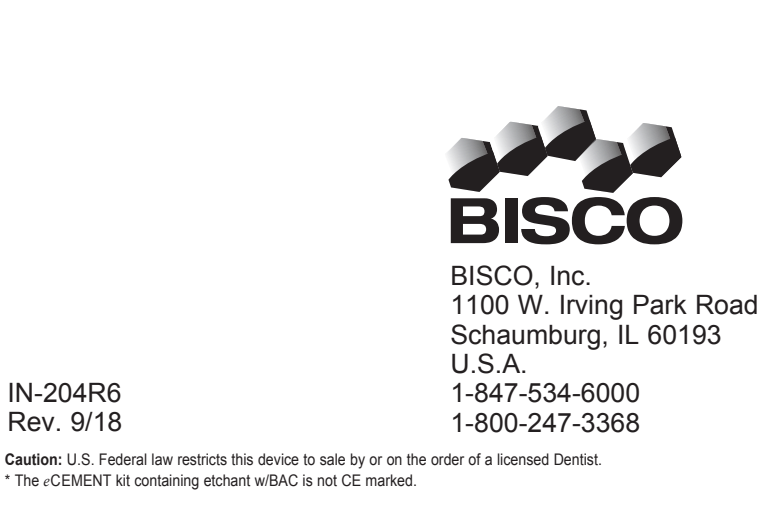

BISCO, Inc.
1100 W. Irving Park Road
 Schaumburg, IL 60193
U.S.A.
IN-20496
Rev. 9/18
1-847-534-6000
1-800-247-3368

Caution: U.S. Federal law restricts this device to be sold by or on the order of a licensed dentist. The CEMENT kit containing etchant wBICO is NOT CE marked.

### CEMENT<sup>®</sup> Adhesive Cmentation System For LITHUM Disilicate Restorations

CEMENT is a resin cementation system designed to simplify the placement of Lithum Disilicate ceramic (i.e. IPS e.max<sup>®</sup>) restorations. Lithum Disilicate is a silica based ceramic in pressed and milled forms, with both high and medium modulus, and high and low translucencies. The CEMENT system contains both light-cure, and dual-cure resin cement, ensuring that all of your Lithum Disilicate restorations will have exceptional retention and esthetics. The CEMENT system may also be used for cementation of other silica based ceramic restorations. CEMENT TRY-IN PASTES are available separately and may be used to visualize and pre-meter the aesthetics of the final cementation. They are water-soluble and easily easy.

Indications for Use:

CEMENT LC: Thin restorations with high translucency, where a light-cured only resin cement is preferred (includes veneers and crown covers)
CMENT DC: Thicker restorations with medium or high opacity and low translucency, where a dual-cured cement is required (includes veneers, high-strength zirconia crowns, and three-unit bridges, primarily in the posterior region)

Contraindications:
Do not use ALL-BOND UNIVERSAL on patients who have a history of severe allergic reactions to methacrylate resins.

Warnings:
ALL-BOND UNIVERSAL and PORCELAIN PRIMER are highly flammable.
Do NOT use PORCELAIN ETCHANT until you have removed the fire hazard.
Hydrofluoric acid is an aggressive chemical. It is a severe eye and tissue irritant. If accidentally splashed into the eye, flush with copious amounts of water for 15 minutes and seek immediate medical attention.

Hydrofluoric acid releases a vapor that irritates respiratory passageways. Chronic inhalation of vapors is dangerous and can cause damage. Extreme care must be taken to protect the patient, operator and assistant. These individuals must wear protective eyewear. The operator and assistant must wear protective gloves. Rubber dam isolation may be used for intentional protection.
Do NOT use PORCELAIN ETCHANT until you have removed the fire hazard.
Use appropriate personal protective equipment.

Very low fumes of ethants prior to applying trinitrate. If resistance is met, re-apply top and re-check. Use only recommended mix. Do not allow any product to be ingested.
Hydrofluoric acid is a very strong irritant. Do NOT use ALL-BOND UNIVERSAL on patients who have a history of severe allergic reactions to methacrylate resins.
Do not clean, deinfected or reuse.

Precautions:
Remove SELECT HV<sup>®</sup> ETCH away from direct sunlight. Prolonged exposure to sunlight could cause a discoloration of the etchant, this will not compromise the efficacy of the product.
Start application of Etchant on a mixing pad or a glass slab. BEFORE use on patients so as to become familiar with the pressure needed to create the Zirconium foam. Use water to increase viscosity before checking flow from top. Never use force to penetrate start flow irregularly, this could result in a subsequent uncontrolled discharge of material. If resistance to flow is experienced, DO NOT proceed. Regulate with new lips and verify flow before use on patient.
PORCELAIN ETCHANT will splash back on the face and gird areas. Such care must be taken to avoid having the etchant contact facial porcelain surfaces unless you desire such splash.
Do not re-use restorations that have been etched by the acid. Over etching porcelain may cause weakening of porcelain or degradation of bond strength.
Prophy pastes containing oil or fluoride should not be used as these additives may interfere with etching.
ALL-BOND UNIVERSAL is a light curable material. Use ALL-BOND UNIVERSAL immediately once it is dispensed in the mixing well. Do not touch the resin with your fingers. Do not use your fingers to clean the application area.
To avoid evaporation of ALL-BOND UNIVERSAL, keep container tightly closed.
BISCO recommends that areas in close proximity to the pulp be protected by application of Thermal-C/LC Light-Cured, Resin-Bonded Calcium Silico-Phosphate (TBC-1) before application of the final restoration. The protractor should be used as a guide prior to taking an impression.
Avoid contact with the skin. Uncopolymerized resins may cause skin sensitization in susceptible persons. In case of contact, wash skin with soap and water.

Working and setting times will be accelerated in the warm clinical environment.
To prevent polymerization or drying, the dual-curing mixing tip should be left in place until the next application. If used intrapatiently, cleaning the mixing tip with disinfectant prior to use on the next patient is advised.
See individual component labels for specific expiration dates.
Do not use hydrogen peroxide, EDTA, or etching containing materials prior to the adhesive procedure using these materials may inhibit curing.
Ensure restoration is prepared per instructions.
Remove the excess cement.
Safety data sheet available at www.Bisco.com

TECHNICAL INFORMATION	
ALL-BOND UNIVERSAL	Light cure for 30 seconds from lingual and 40 seconds from the facial, at a distance of approximately 0-10 mm.
Dual-Cure Resin Cement	Working Time: Minimum 2 minutes at 22°C (71.6°F). Setting Time: Maximum 3 minutes, 30 seconds at 37°C (98.6°F)*
ALL-BOND UNIVERSAL	Light cure for 30 seconds
Dual-Cure Resin Cement	Working Time: Minimum 2 minutes at 22°C (71.6°F). Setting Time: Maximum 3 minutes, 30 seconds at 37°C (98.6°F)*
ALL-BOND UNIVERSAL	Light cure for 30 seconds
Dual-Cure Resin Cement	Working Time: Minimum 2 minutes at 22°C (71.6°F). Setting Time: Maximum 3 minutes, 30 seconds at 37°C (98.6°F)*
ALL-BOND UNIVERSAL	Light cure for 30 seconds
Dual-Cure Resin Cement	Working Time: Minimum 2 minutes at 22°C (71.6°F). Setting Time: Maximum 3 minutes, 30 seconds at 37°C (98.6°F)*

\* Working time and setting time may vary based on storage conditions, temperature, humidity, etc

**WORKING AND SETTING TIME** may vary based on storage conditions, temperature, humidity, etc

### INSTRUCTIONS FOR USE

**STEP 1. Restoration Preparation and Try-In**
1. Thoroughly clean the restoration, remove cap from PORCELAIN ETCHANT, scrubbing against black splash dispensing tip and verify etchant flow prior to application. Apply PORCELAIN ETCHANT to the bonding surface, 25 seconds for Lithum Disilicate (or contact laboratory for recommended etch times for other ceramics). Rinse and dry.
2. Apply a coat of PORCELAIN PRIMER to the internal surface of the restoration and allow to flow for 30 seconds. Dry with air blowing for 3-5 seconds.

**CEMENT TRY-IN PASTE**
Apply the desired shade to the silica - coated restorations and gently seal on the teeth.
NOTE: THE TRY-IN PASTE is not intended for use on the preparation of the restoration.

After try-in, decontaminate the internal surface of the restoration (photostrict acid) or 2 minutes ultrasonic cleaning with ethanol using proper contamination while leaving the PORCELAIN PRIMER intact. Rinse and air dry.

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**WORKING AND SETTING TIME** may vary based on storage conditions, temperature, humidity, etc

Do not use ALL-BOND UNIVERSAL on patients who have a history of severe allergic reactions to methacrylate resins.

Hydrofluoric acid releases a vapor that irritates respiratory passageways. Chronic inhalation of vapors is dangerous and can cause damage. Extreme care must be taken to protect the patient, operator and assistant. These individuals must wear protective eyewear. The operator and assistant must wear protective gloves. Rubber dam isolation may be used for intentional protection.
Do NOT use PORCELAIN ETCHANT until you have removed the fire hazard.
Use appropriate personal protective equipment.

Very low fumes of ethants prior to applying trinitrate. If resistance is met, re-apply top and re-check. Use only recommended mix. Do not allow any product to be ingested.
Hydrofluoric acid is a very strong irritant. Do NOT use ALL-BOND UNIVERSAL on patients who have a history of severe allergic reactions to methacrylate resins.
Do not clean, deinfected or reuse.

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