BISCO

CE 0459

BARRIER GEL

Etchant Protectant

Instructions for Use

BISCO, Inc.
1100 W. Irving Park Rd.
Schaumburg, IL 60193
U.S.A.
1-847-534-6000
1-800-247-3368

IN-153R5
Rev. 6/20

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed Dentist.

en **BARRIER GEL***
Etchant Protectant

GENERAL INFORMATION

BISCO **BARRIER GEL** may be placed to protect areas where etching is not desired. It may also be used to protect a small area of tissue that may still be exposed after the rubber dam is placed.

Indications for Use

Protect areas where etching is not desired.

Warning:

- Do not use hydrofluoric acid etchant until you have thoroughly read and understand those instructions.

Caution:

- Cross-contamination:
 - Product may contain items that are designed for one time use. Dispose of used or contaminated tips. Do not clean, disinfect or reuse.
 - Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If syringe becomes contaminated, discard. Do not clean or disinfect.

Precautions:

- Safety data sheet available on request.
- Safety data sheet available at www.bisco.com.

INSTRUCTIONS FOR USE

NOTE: Care must be taken to avoid having porcelain etchant contact glazed porcelain surfaces since these surfaces will lose their gloss.

- Isolate the area to be repaired with a rubber dam.
- Place **BARRIER GEL** on gingival tissue that may be exposed or on porcelain to protect areas where etching is not desired.
- After etching according to manufacturer instructions, suction the porcelain etchant along with the **BARRIER GEL** to remove. Rinse with a copious amount of water with suction and air dry.

HYGIENE: Re-cap syringes with the luer lock cap. Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended.

DISPOSAL: Refer to community provisions relating to waste. In their absence, refer to national or regional provisions relating to waste.

STORAGE: Store at room temperature (20°C/68°F - 25°C/77°F).

WARRANTY: BISCO, Inc. recognizes its responsibility to replace products if proven to be defective. BISCO, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

*BARRIER GEL is manufactured by BISCO, Inc.

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* BARRIER GEL é fabricado pela BISCO, Inc.