Bisco

CE

ACE[®]  **Dispenser**
one and done

Instructions For Use

U.S. Patent 7,748,980


IN-179R2

Rev. 9/12

BISCO, Inc.
1100 W. Irving Park Rd.
Schaumburg, IL 60193
U.S.A.
847-534-6000
1-800-BIS-DENT

ACE®* DISPENSER INSTRUCTIONS FOR USE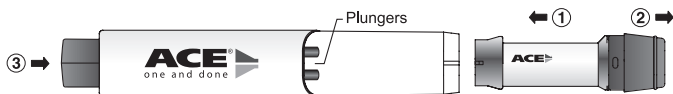

▶ TO LOAD AND ACTIVATE THE NEW CARTRIDGE

NOTE: Ensure the two red plungers of the ACE Dispenser are fully retracted by pressing the eject button. (Plungers should not be visible prior to loading cartridge.)

1. **With label facing up**, insert cartridge into the ACE Dispenser.
2. Remove cap.
3. Press orange button repeatedly until drops dispense from both tips of the cartridge.

▶ TO DISPENSE PRODUCT FOR USE

1. Position both cartridge tips over a mixing well and press orange button 1 time. The correct proportion of material will be dispensed. If needed, touch the tips to the side of the mixing well to release the drops.
2. Replace cap tightly on the cartridge. **It is recommended the cartridge be left in the ACE Dispenser until depleted.**

▶ TO EJECT THE CARTRIDGE

1. To remove cartridge, turn the ACE Dispenser so the logo is facing down. **With cap removed**, slide the eject button.
2. Pull the cartridge out of the ACE Dispenser.
3. Discard cap and cartridge.
4. Load a new cartridge.

DISINFECTION: Dispenser – Use surface disinfectant spray or wipes recommended for ABS, PVC and polycarbonate (PC) plastics according to manufacturer's instructions. Avoid direct contact of disinfection solution with the adhesive material at the cartridge tips. Do not submerge or autoclave.

Mixing Well – Soak in a cold chemical bath materials compatible with polypropylene according to manufacturer instructions.

WARRANTY: BISCO, Inc. recognizes its responsibility to replace products if proven to be defective. BISCO, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

* ACE is a registered trademark of BISCO, Inc.

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Made in the U.S.A.