BISCO

Unit-Dose Syringe

Instructions for Use

 BISCO, Inc.
1100 W. Irving Park Road
Schaumburg, IL 60193
U.S.A.
1-800-247-3368
1-847-534-6000
www.bisco.com

IN-210R2
Rev. 12/20

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed Dentist.

en**UNIT-DOSE SYRINGE**

BISCO's Unit-Dose Syringe dispenser is designed to easily dispense composite, cement and flowable materials with precision. The Unit-Dose Syringe Dispenser can be used with any of BISCO's prefilled unit-dose tips.

Intended user: licensed dentist.

Precautions:

- If used intraorally, sterilize the Unit-Dose Syringe dispenser according to instructions prior to storage.
- Instructions for Use available upon request.
- Instructions for Use available at www.bisco.com.

INSTRUCTIONS FOR USE

1. Place tube or compute into the syringe barrel. Remove cap.
2. Squeeze syringe with a slow, steady pressure.
3. Remove and dispose tube or compute after use.

STERILIZATION:

Clean the dispenser prior to sterilization: Lift rear handle up to allow removal of black plunger. Remove metal spring from plunger and place all three parts (syringe, plunger and spring into ultrasonic cleaning unit, cover and clean for 16 – 20 minutes (at 122F/50C if unit contains a heater). Remove and rinse thoroughly under sterile fresh warm water (de-mineralized water may be used). Shake to remove excess water and stand vertically to air dry. The syringe should be dry before sterilization. Reassemble spring onto plunger making sure small spring hold is at back of plunger for proper reassembly.

Bag and wrap in an autoclavable wrap, seal, and arrange in Autoclave. Proceed with a gravity cycle sterilization procedure according to the Sterilizer Manufacturer's instructions; 20 minutes at 250F/121C- 15psi.

DISPOSAL: Sterilize per instructions and dispose in waste receptacle.

WARRANTY: BISCO, Inc. recognizes its responsibility to replace products if proven to be defective. BISCO, Inc. does not accept liability for any damage or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

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